


Instruction for FPV Ag test



The amount of sample : 100 μ l

1 Sampling

- 1. Do not use sticky or a lump of feces.
 - 2. Don't collect too much feces.
- Maximum quantity : 30 mg
(Fig : Swab with 30 mg of feces)

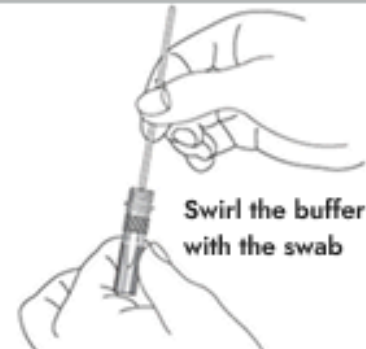

Swirl the buffer with the swab

2 Use the Supernatant layer



Wait until feces particles sink to the bottom



Supernatant layer for use

Feces particle layer




3 Loading

4 drops (100 μ l) of diluted sample
⚠ Don't overload the sample



4 Reading

00:05:00-00:10:00

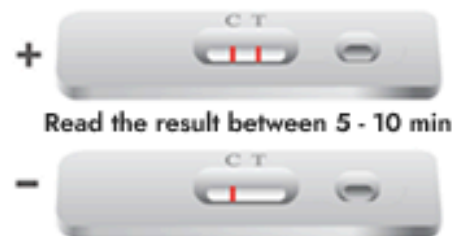

Read the result between 5 - 10 min

5 Discard

00:10:00



Discard the used device
Invalid result after 10 min

